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**INSTRUCTIONS FOR USE****EN****DESCRIPTION**

The Drentech™ Chest system is a single chamber thoracic drainage system characterised by a one-way water valve related directly to the collection chamber enabling to visualize immediately patient's air losses. This device is available in two versions: the first one is characterised by a fixed and integrated pillar, connecting drainage entrance to the bottom of the collection chamber. In this way the exit distal position is always unchanged. In the second version, the above mentioned pillar is carried out directly to maintain unchanged its distance from the level of the drained fluid.

The system is composed of the following parts:

1. A collection chamber (1) which has a max capacity of 2250ml.
2. A connector (2) for the preliminary filling of the system in order to carry out the on-way water valve. This connector is provided with a special sealing cap (9).

- Patient connection tube (3) provided with a suitable tapered connector (4) for the connection to drainage catheters.
- In the version with fixed fluid exit position, the patient connection tube is related to a pillar (5) integrated in the collection chamber. In this pillar is possible to visualise the fluid oscillations which are synchronous with patient breathing movements. In the version with changeable fluid exit position, the pillar (5) is characterised by the same patient connection tube (3), whose end can be adjusted in height as required. Also in this case, it is possible to visualise the fluid oscillations which are synchronous with patient breathing movements.
- The system is provided with a connector (6), to evacuate patient air and to connect it to a 3-Mobile Chest Vacuum Unit, and with a closed support (7) to fix the unit.
- Moreover, the drainage system is provided with a handle (8) for transporting and fixing the system to the bed and with suitable I.V. pole and with a closure cap (11) to facilitate the disposal of the system after its use.

**NOTE:** To connect the Drentech™ Chest to a 3-Mobile Vacuum Unit it is necessary to use the suitable sterilizing filter (10) which is available separately.

#### INSTRUCTIONS FOR USE

The Drentech™ Chest drainage system is suitable for: evacuating air and/or fluid from the chest cavity or mediastinum; preventing air and/or fluid accumulation in the chest cavity; re-establishing and maintaining normal gradients of intrathoracic pressure; facilitating complete pulmonary expansion to restore normal respiratory dynamics. This device is mainly recommended for the use by suction connected to a 3-Mobile Chest Vacuum Unit that provides a movable and adjustable vacuum source. This device can also be used by gravity.

**Note:** This device can only be used in safety conditions in the field of applications and using the procedures indicated on this operating instruction sheet for the specific type of product. The manufacturer declines all liability relating to improper use or use different to that indicated.

#### WARNINGS/PRECAUTIONS

- The drainage system must be maintained in vertical position, also during transport and always below the patient level (at least 50 cm).
- Avoid loops or squashing the patient drainage tube (3).
- During the use by gravity, do not obstruct or cover the connector (6) to allow patient air evacuation.
- Thanks to its particular design, the system is stable and does not require an I.V. pole for use on the floor.
- The support (7) is a closed support which has the only function to allow the fixing of the Chest Vacuum Unit.
- In the version with changeable fluid exit position, it is necessary to check the lower end is always submerged into the fluid for at least 2 cm. The exposure of the lower end of the pole allows air flux towards the patient and can be the cause of pneumothorax.

#### GENERAL WARNINGS

Single-use product. Reuse may lead to alteration of performance and risks of cross-contamination. Do not use if the package has been opened or damaged.

#### PREPARING THE SYSTEM FOR USE

Extract the device from its housing using an aseptic procedure. Carry out the preliminary filling of the collection chamber using sterile water according to the conditions and quantities described in Figure A (a quantity of water equal to 350ml is required). After the filling, close the suitable cap (09). As in both versions the patient connection tubes are already assembled, after filling the system is ready for use so that it is possible to connect the drainage catheter to the suitable connector.

Now, positioning the system under the patient level, the chest drainage starts by gravity. In both versions, near the end of fluid exit, it is possible to visualise possible patient air losses thanks to the appearance of more or less concentrated boiling. This visualization is facilitated by the frontal position of the end of exit fluid and by the transparency of the front side of the collection chamber.

#### OPERATING BY SUCTION

In order to use the drainage system by suction, it is possible to connect the Chest Drentech™ system to a Drentech™ Chest Vacuum Unit. To carry out this operation, always refer to the respective instructions for use manual.

#### CONTRAINDICATIONS

No known contraindications exist for the use of this device. The choice between the drainage by gravity or by suction must be decided by an expert physician on the basis of clinical needs.

#### STERILE - Ethylene oxide sterilized

The product is sterile if the package is intact. Discharge after each single use, do not re-use. Avoid exposure to high temperatures and ultraviolet rays during storage. For discharging and disposal of the

device, adopt suitable precautions and act in accordance with the provisions of the law in force for biologically hazardous waste. The sealing cap, which is included, can be used to close the connector (6) before disposal.

**Note:** the fluid contained in the collection chamber of the drainage system must not be used for reinfusion.

#### MATERIALS USED

PS, PVC, and PP.  
LATEX FREE MEDICAL DEVICE.  
PVC FREE PACKAGING

Date of issue of the last version:  
see last page : (REV.: XX-XXXX)

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