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INSTRUCTIONS FOR USE

DESCRIPTION

The Redon surgical drain is a sterile single-use device. Various sizes are available supplied in a double sterile package. Versions are available with the Trocar needle already fitted to the drain to facilitate positioning with an aseptic technique. All the drains have depth markers, graduated from the end of the perforated section.

INDICATIONS

The surgical drains are used in numerous and different fields of application for the purpose of evacuating potentially pathological fluids and blood that have built up in natural and/or neoforned cavities following surgery. All the surgical drains are compatible for use in association with postoperative blood recovery systems in order to perform autotransfusion.

SUGGESTIONS FOR USE

Positioning the drain

- Before positioning the drain, carefully select the type, the position and the required size. These parameters depend on the type of operation, the physiology of the patient and the experience of the surgeon.
- Using an aseptic technique, carefully take the sterile chest drain tube out of the protective wrapping.
- Position the drain in the wound following an aseptic procedure. For drains equipped with a Trocar needle, use it according to the usual positioning technique working from inside the wound to the outside.
- Use the depth markers indicated on all the drains for correct positioning.
- Secure the drain with a skin stitch or a plaster.
- Connect the positioned tube to the chosen drainage system

according to the procedures indicated in the relative instruction leaflet. In the case of drains equipped with a Trocar needle, cut it and dispose of it before connecting to the drainage system.

Removing the drain

1. Remove the tape or stitch or whatever was used to secure the drain.
2. Disconnect the drainage system.
3. Remove the drain by pulling it gently and immediately apply a dressing on the wound.

WARNINGS/PRECAUTIONS FOR USE

1. **Do not in any way handle the drain or make extra holes with the aid of cutting tools.**
2. **If securing the drain with a stitch, be extremely careful not to suture the tube or cut it with needles or sharp tools. Do not over-tighten the suture as this might cause the drain to break.**
3. **After positioning the drain, check that the perforated section is fully inserted in the wound and that the connections are perfectly sealed.**
4. **Manually remove the drain pulling it gently avoiding abrupt manoeuvres. Do not use metal instruments, such as forceps or devices that may cause the drain to break.**
5. **If the drain is left in place for very long periods of time, it may be difficult to remove. Be extremely careful during the removal operations.**
6. **Once the drain has been removed, check that it is fully intact, an essential condition to exclude that drain fragments have been left in the cavity caused by accidental rupture.**
7. **It is recommended not to use the drain for more than 29 days.**
8. **Do not use roller clamps or other metal devices to milk to silicone tubes.**
9. **Do not use abrasive materials, such as wads of cotton or gauze. Milking the silicone tubes using this type of material may cause damage to the drain surface reducing its mechanical resistance.**
10. **Do not use any alcoholic or aggressive liquid to lubricate the tube surface before milking. Redax strongly suggests using a cream or Vaseline oil or any other non-aggressive lubricant to help remove the tube.**

CONTRAINDICATIONS

The physician has to assess whether there are any clinical conditions that would make drain positioning inadvisable (e.g. presence of fistulas in the internal organs, tissue necrosis, osteomyelitis, etc.).

GENERAL WARNINGS

The product is sterile if the package is intact. Do not use if the package is open or damaged. Avoid exposure to high temperatures and ultraviolet rays during storage.

This device can be used in safe conditions only in the field of application and according to the procedures indicated in this instruction leaflet in correspondence to the type of product. The manufacturer declines all responsibility for improper use or different from that indicated.

DISPOSAL

Adopt adequate precautions for elimination and disposal of the device and comply with the provisions of the laws in force on biologically hazardous waste.

SINGLE-USE DEVICE

Reuse may lead to alterations in performance and cross-contamination risks.

STERILE - Sterilized by ethylene oxide

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